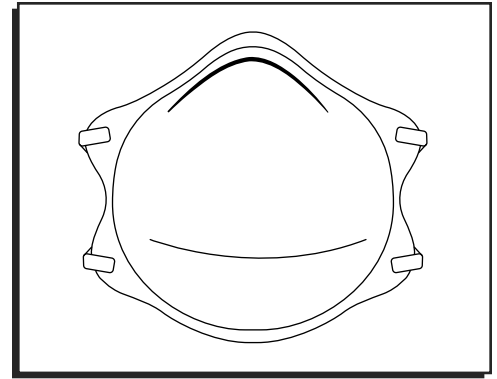
ULINE S-26497, S-26498 BLACK N95 RESPIRATOR

1-800-295-5510
uline.com

SAFETY

- Wearer must read fitting instructions and use limitations before use.
- This respirator is not guaranteed to protect against the risk of contracting a disease or infection.
- Before occupational use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training and fit testing.
- Do not use the same respirator for more than 8 hours, continuous or intermittent, in dirty workplaces that could result in high filter loading.
- Leave contaminated area and remove respirator if breathing becomes difficult or proper fit cannot be obtained or the respirator is damaged, soiled or distorted. Do not use the same respirator again once it has been used in contaminated area.
- The maximum service time of this respirator is five (5) consecutive calendar days (including days of non-use), beginning from the first day of use.

SPECIFICATIONS

THIS RESPIRATOR IS APPROVED ONLY IN THE FOLLOWING CONFIGURATION:

TC-	PROTECTION ¹	RESPIRATOR	CAUTIONS AND LIMITATIONS ²
		S-26497	
84A-7882	N95	X	ABCJMNOPS

TC-	PROTECTION ¹	RESPIRATOR	CAUTIONS AND LIMITATIONS ²
		S-26498	
84A-7883	N95	X	ABCJMNOPS

1. PROTECTION

N95 Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.


2. CAUTIONS AND LIMITATIONS

- Not for use in atmospheres containing less than 19.5% oxygen.
- Not for use in atmospheres immediately dangerous to life or health.
- Do not exceed maximum use concentrations established by regulatory standards.
- Failure to properly use and maintain this product could result in injury or death.
- All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
- Never substitute, modify, add or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- Refer to user's instructions and/or maintenance manuals for information on use and maintenance of these respirators.
- NIOSH does not evaluate respirators for use as surgical masks.
- Special or critical user's instructions and/or specific use limitations apply. Refer to user's instructions before donning.

FITTING INSTRUCTIONS


1. Hold the respirator in hand with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand. (See Figure 1)

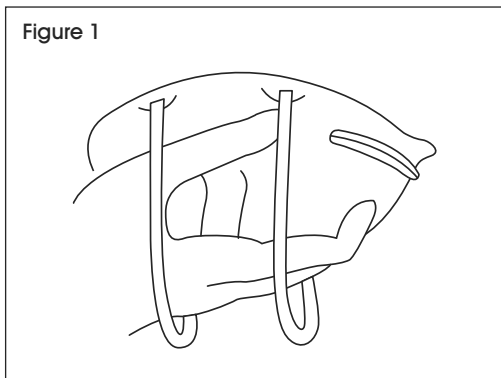Figure 1


2. Press the respirator firmly against your face with the nosepiece on the bridge of your nose. Stretch and position the top band high on the back of the head. Stretch the bottom band over the head and position below your ears. (See Figure 2)

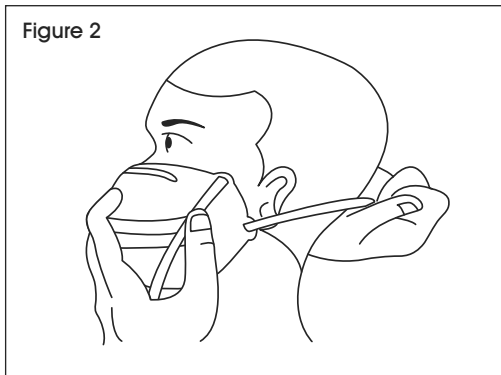Figure 2

3. Using both hands, mold nosepiece to the shape of your nose. (See Figure 3)

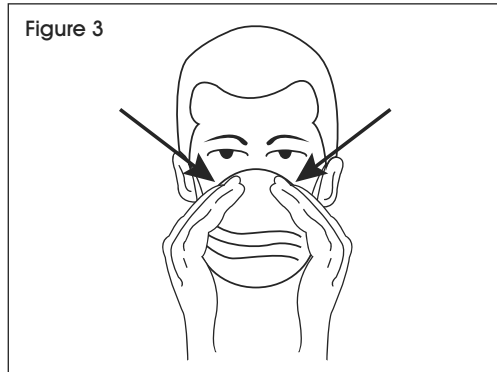Figure 3


4. To test fit, cup both hands over the respirator, being careful not to disturb position, and inhale vigorously. If air leaks around the edges, reposition the straps or adjust strap tension where applicable for better fit. (See Figure 4)


NOTE: Follow fitting instructions before each use to achieve proper fit.

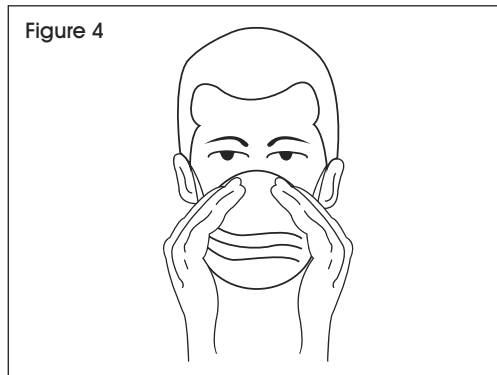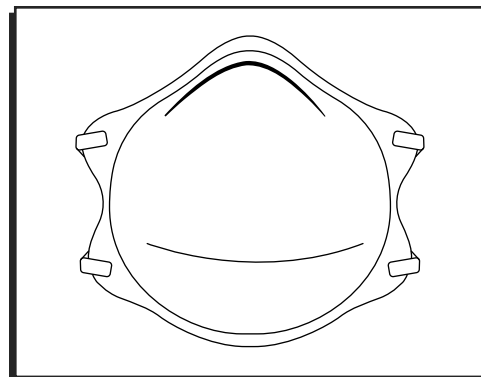Figure 4

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