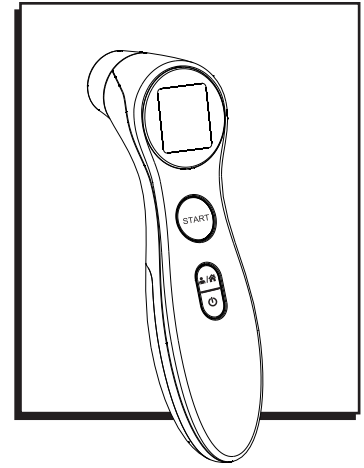
**ULINE H-8862**  
**INFRARED FOREHEAD**  
**THERMOMETER**

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**TROUBLESHOOTING**

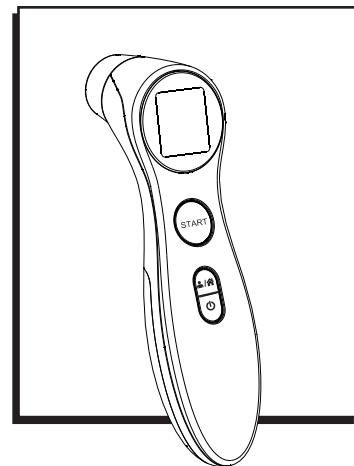
ERROR MESSAGE	PROBLEM	SOLUTION
<i>Er1</i>	Measurement before thermometer is ready	Take a measurement until the forehead mode icon  or object mode icon  appears on the display.
<i>Er2</i>	The ambient temperature is not within the range between 50°~104°F.	Place thermometer in a room for at least 30 minutes at room temperature between 50°~104°F.
<i>Er3</i>	The thermometer is placed incorrectly or unsteady.	Read instructions thoroughly and take a new temperature measurement.
<i>Er4</i>	The thermometer is showing a rapid ambient temperature change.	Allow the thermometer to rest in a room for at least 30 minutes at room temperature: between 50°~104°F.
<i>Er5</i>	The thermometer is not functioning properly.	Unload the battery, wait for one minute and repower it. If message appears, contact Uline Customer Service at 1-800-295-5510.
<i>Hi</i>	In Forehead Mode: Temperature taken is lower than 109.4°F.  In Object Mode: Temperature taken is lower than 212°F.	Read Temperature Taking Hints thoroughly, then take the new temperature measurement. (see page 3 of <a href="#">instructions</a> )
<i>Lo</i>	In Forehead Mode: Temperature taken is lower than 93.2°F.  In Object Mode: Temperature taken is lower than 32°F.	Read Temperature Taking Hints thoroughly, and ensure the lens and filter are clean. Take the new temperature measurement. (see page 3 of <a href="#">instructions</a> )

## TROUBLESHOOTING CONTINUED

ERROR MESSAGE	PROBLEM	SOLUTION
	The thermometer does not work due to low battery.	Replace old batteries with two new AAA alkaline batteries.

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